Criorex® Emulgel ematomi e traumi

IT

CRIOREX® EMULGEL

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EN

CRIOREX® EMULGEL HEMATOMA AND TRAUMAS

CE Medical Device CREAM with B.T.E. Hyperthermal Water (Euganean Thermal Basin) extracted from therapeutic springs.

Instructions for use

CRIOREX® Emulgel is indicated to relieve joint and muscle pain due to bruises, strains, dislocations, sprains and traumas caused by physical and sporting activity. The application of CRIOREX® Emulgel produces an instant cooling effect on the skin surface.

FAST ACTION – Gel based on hyperthermal salso-bromo-iodic water, disodium EDTA, sodium magnesium silicate, alcohol and ozonated oil. It also contains other components that perform a moisturizing and soothing action on the skin. The application of CRIOREX® Emulgel produces an instant refreshing effect on the surface of the skin, with a pleasant minty fragrance. Thanks to its composition it facilitates the natural process of absorption of oedema, it is indicated in first aid, to relieve swelling and pain and relieve the affected area.

Components

Light, smooth and free of parabens, the exclusive formula of CRIOREX® Emulgel contains:

- sodium magnesium silicate – a clay with a draining effect
- euganean Hyperthermal Water – extracted from one of the deepest geothermal wells in the area and rich in beneficial properties that physiologically help restore the trophism of cartilages and reduce phlogistic phenomena in the physiatriac field
- denatured ethyl alcohol – with an astringent cold effect
- ozonated oil – that draws oxygen and promotes cell

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INFO@THERMALIS.IT — (+39) 041 0996 863

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regeneration

- Disodium EDTA – iron chelator that allows to attenuate the bluish color of ecchymosis

To guarantee the product integrity, a Coex tube has been used as an additional protective barrier.

How to use

Apply a thin layer of cream-gel 2-3 times a day on the painful area. Massage gently until completely absorbed.

Warnings

Store in a cool and dry place, away from sunlight and heat sources. Keep out of reach of children. Discontinue use in case of irritation. Do not use on broken skin, open wounds or on people with skin diseases. Do not use on children under the age of 6. For external use only. Do not swallow. Avoid contact with eyes and mouth; in case of accidental contact, rinse thoroughly with water. Do not use in case of known hypersensitivity to the components. If pregnant and/or breastfeeding, consult your doctor before use. Do not use on newly formed skin. Do not exceed the recommended dose. The expiry date refers to the intact and well preserved product. The manufacturer declines all responsibility for damage to the user caused by improper use and any use different from that indicated on this information leaflet. Do not use after the expiration date. Wash your hands after use. Store in a cool and dry place, away from sunlight and heat sources, at a temperature between 4 and 30 degrees.

Packaging

Tube 100 ml.

Ingredients

Aqua*, Alcohol denat., Isopropyl alcohol, t-Butyl alcohol, Urea, Polyacrylamide, C13-14 isoparaffin, Laureth-7, Sodium magnesium silicate, Xanthan gum, Citric acid, Menthyl lactate, Aqua, PPG-25-laureth-25, PEG-40 hydrogenated castor oil, Glycerin, Xylitol, Erythritol, Menthol, Coco-caprylate, Tocopherol, Ozonized olive oil, Disodium EDTA, Citric acid, Anthocyanins, Methylglucamine.

*With Salso-bromo-iodic Hyperthermal Water

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