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EN

ISOTONIC VIALS UPPER RESPIRATORY TRACT

HYPERTHERMAL WATER from B.T.E. (Euganean Thermal Basin) extracted from therapeutic springs.

General information

Hyperthermal Salt-Bromine-Iodic Water from the Euganean Thermal Basin is rich in sodium, chlorine, bromine, iodine and many other mineral elements that are fundamental in biochemical processes.

ORIGINALITY AND UNIQUENESS OF THE PRODUCT - THERMALIS®

Hyperthermal Water is extracted from one of the deepest existing therapeutic hot springs in the Euganean Region, and is used as a basic ingredient in all of the products. Its osmotic effect restores the physiological state of the mucous membranes.

THERMALIS® Hyperthermal Water is available in vials and no-gas sprays.

VIALS RESEALABLES - MEDICAL DEVICE CE 789367/R with Hyperthermal Water from the Euganean Thermal Basin as a base ingredient. Isotonic solution (0,9 %).

Hyperthermal Water is indicated for reducing upper respiratory tract disorders. Thermal treatments for ear, nose and throat infections contribute physiologically to the:

- elimination and thinning of nasal secretions
- protection of nasal mucous membranes against irritation caused by pollution
- decongestion of the nose
- stimulation of nasal cilia
- normalisation of respiratory mucous
- humidification of the mucous membranes

This product can also be used with nasopharyngeal aerosol devices. Preservative free. The product has no pharmacological or metabolic action and is natural and physiological. Water made isotonic by the addition of sodium chloride. Can also be used to treat children.

How to use

The nasal rinse can be carried out easily and safely either manually or with a nebuliser. Shake before use.

Manual application:

- open the vial with a twisting motion
- slide the applicator gently into the nasal cavity
- press the vial and inhale
- let the product act for a few seconds and repeat for the other nostril

With nasopharyngeal aerosol device:

- open the vial with a twisting motion
- insert the contents of the vial into the nebuliser, following the manufacturer's instructions

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INFO@THERMALIS.IT — (+39) 041 0996 863

If you wish to reseal the vial, turn the stopper upside down and press it into the opening. Once opened, the vials must be used within 12 hours and stored as specified under warnings. The dosage and duration of the treatment are recommended by a specialist or family doctor.

Warnings

Store in a cool, dry place, away from direct heat or sunlight. After using, close with the supplied stopper. Keep out of reach of children. Suspend use in the event of irritation. Do not use if you are allergic to any of the ingredients. The expiry date refers to the product when it is intact and stored correctly. The manufacturer accepts no responsibility for damage to the user caused by improper and / or different use from that indicated in the leaflet. Do not use after the expiry date. Do not ingest.

Packaging

Blister pack with 15 resealable 5 ml vials.

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