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EN

## PHYSIOREX® CREAM MUSCLE AND JOINT PAIN

CE 770601/R MEDICAL DEVICE with Hyperthermal Water from the B.T.E Euganean Thermal Basin.

FAST ACTION - It is a valuable aid in the relief of pain involving ligaments, tendons, soft tissues. Recommended before practicing sports to optimise the warming up stage and recovery.

ADJUNCT TO PAIN RELIEF AND MUSCLE RELAXATION TREATMENTS

A muscle relaxation and pain relief massage is an ancient therapy that aims to act beneficially on:

- muscular tension, also in the form of cramps and muscle spasms
- painful involuntary and persistent muscle contractions
- locked joints
- fibromyalgia/muscular rheumatism
- muscle-tension pains in the neck and shoulders
- sports injuries
- accumulation of lactic acid after sports activities

Also ideal before practicing sports to optimise the warming up stage.

The exclusive formula of PHYSIOREX®, a paraben and petrolatum-free light and easy to apply cream, contains:

- Selected Active Ingredients of plant origin (including *ECOCERT Physalis Angulata*), which among their beneficial effect include mild heat and a sensation of pain relief without skin redness or irritation
- Euganean Hyperthermal Water, extracted from one of the

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INFO@THERMALIS.IT — (+39) 041 0996 863

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deepest existing geothermal wells in the area and well-known for its ability to assist in the physiological processes for restoring trophism in the cartilage and relieving nerve, muscle, and bone disorders

#### How to use

Apply a thin layer of cream to the problem area 2 (6-11 years old) or 3 times (12+) a day, massaging gently until it is fully absorbed.

#### Warnings

Store in a cool, dry place, away from direct heat or sunlight. Keep out of reach of children. Suspend use in the event of irritation. Do not use on damaged skin, open wounds. External use only. Do not use on children under 6 years old. Do not ingest. Avoid contact with eyes and mouth; in the event of contact rinse thoroughly with water. Do not use if you are allergic to any of the ingredients. Do not use on newly formed skin. Do not exceed the recommended dosage. The expiry date refers to the product when it is intact and stored correctly. The manufacturer accepts no responsibility for damage to the user caused by improper and/or different use from that indicated in the leaflet. Do not use after the expiry date. Wash hands after use.

#### Packaging

Tube 100 ml.

#### Ingredients

Aqua\*, Ethylhexyl Palmitate, Dicaprylyl Ether, Cetearyl Alcohol, Glyceryl Stearate, Cetearyl Glucoside, Caprylic/Capric Triglyceride, Phytosterols, Tocopherol, Sodium Polyacrylate, Phenoxyethanol, Ethylexyglycerin, Vanillyl Butyl Ether, Pentaerythrityl tetra-di-t-butyl hydroxyhydrocinnamate.

\*With Salso-bromo-iodic Hyperthermal Water

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