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Acqua Ipertermale Salsobromiodica, Sodium Chloride F.U.

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EN

HYPERTONIC SPRAY NASAL DECONGESTION RELIEF

Hyperthermal Water from B.T.E. (Euganean Thermal Basin) extracted from therapeutic springs.

General information

The Salso-bromo-iodic Hyperthermal Waters of the B.T.E. (The Euganean Thermal) Basin contains sodium, chlorine, bromine, iodine and other important trace elements essential for biochemical processes.

ORIGINALITY AND UNIQUENESS OF THE PRODUCT - THERMALIS® Hyperthermal Water is extracted from one of the deepest therapeutic wells existing in the Euganean territory and added as a basic element of all products. Its osmotic effect restores the physiological state of the mucous membranes.

THERMALIS® packages its Hyperthermal Water in bottles and sprays without gas.

SPRAY FOR NASAL WASHES - CE MEDICAL DEVICE based on Hyperthermal Water from B.T.E. Euganean Thermal Basin. Hypertonic solution 3% that effectively decongests, relieves stuffy noses and facilitates breathing during a cold, allergic rhinitis, whether in the acute or chronic phase. Free of preservatives and propellants gas, this product offers a physiological and natural solution to the afore-mentioned ailments.

It is administered easily thanks to a spray directly in the nostril to ensure the permeability of the nasal cavities. Its formula is used to decongest the nose, moisturize and cleanse the mucous membranes. It is also useful in case of sinusitis.

The solution has a high concentration of salts (3%), which provides hypertonic properties. This characteristic causes a pressure difference between the solution and the mucous membranes (osmotic effect), therefore promoting the drainage of liquids and relieving nasal congestion.

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INFO@THERMALIS.IT — (+39) 041 0996 863

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Nasal irrigations with Salso-bromo-iodic Euganean Waters have many positive effects:

- Remove mucus, crusts and other debris that may be present in the nasal passages
- decongest the mucous membranes while moistening them, thus promoting the restoration of their permeability and adequate ventilation of the paranasal sinuses
- promote improved mucociliary clearance, i.e. the movement of microcilia and the normalized transport of mucus from the paranasal sinuses to the nasopharynx
- Mechanically eliminate allergens
- Reduce swelling of the nasal mucosa in colds, rhinosinusitis and allergies
- Clean the nasal cavities
- Help reduce pressure in the nasal passages
- may also help reduce the use of decongestant medications
- Can be used in addition to other nasal treatments
- Are non-addictive
- Can be used during pregnancy and lactation
- Can be used in children over three years old
- Are indicated for washing the nasal cavities of smokers and those who work in contact with polluting substances and/or fine dust

How to use

Keeping your head straight and the spray in a vertical position, insert the nasal spray tip into a nostril. Press the valve and simultaneously inhale through your nose. Repeat the operation in the other nostril. Shake before use. Rinse the dispenser after use.

Dosage

ADULTS

Perform 2 to 3 sprays per nostril 2 to 3 times a day. Let the solution act for a few seconds, then blow your nose, if necessary.

CHILDREN

1 spray in each nostril up to 3 times a day for children aged 3 to 12, 1 or 2 sprays in each nostril up to 3 times a day for children aged 12 and over.

Warnings

Do not use in case of hypersensitivity to any of the components. Do not use if the package or bottle appears damaged. Do not swallow; avoid contact with eyes and wounds; in case of accidental contact, rinse thoroughly with cold water.

Do not use after the expiry date indicated on the packaging.

Use the product only for the purpose for which it is intended. Keep out of the reach of children.

Do not dispose of the container in nature after use. Store in a cool, dry place away from direct heat sources; do not expose to direct sunlight.

Side effects

Use of the device may cause rare events of a benign and transient nature such as: irritation, itching or burning. Immediately discontinue use of the product in the event of such events or any event not listed above and, if symptoms persist, consult your doctor.

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