Thermagyn® lavanda vaginale

IT

THERMAGYN® LAVANDA VAGINALE

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EN

THERMAGYN® VAGINAL DOUCHE

THERMAGYN® - MEDICAL DEVICE for vaginal douche with Hypothermal Water from B.T.E. Euganean Thermal Basin (extracted from therapeutic springs).

THERMAGYN® Vaginal Douche uses the natural soothing properties of Salsobromiodic Hypothermal Water and chamomile floral water to promote delicate vaginal hygiene.

Non-sterile medical device that promotes the rebalancing of the natural bacterial flora by helping to restore the physiological pH of the vagina.

Indications

THERMAGYN® Vaginal Douche is recommended for delicate feminine intimate hygiene, without altering the balance of the vaginal flora. It promotes the elimination of organic residues or previously used medications. THERMAGYN® is indicated as an adjunct in the treatment of irritation, burning and itching associated with dryness in the vaginal cavity, also due to menopause.

It can be used as an adjuvant in the treatment of gynaecological infections (vaginitis of various aetiologies) affecting the female genital organs.

It can also be used for intimate hygiene, at the end of the menstrual period.

How to use

For effective cleansing, apply for three consecutive nights at the end of the menstrual cycle and whenever needed.

As an adjuvant in the treatment of inflammation of the vagina, please follow the instructions of the medical specialist.

Use the entire bottle as follows:

1. check the integrity of the packaging, bottle and cannula
2. wash your hands before handling the product
3. push the top of the cap to one side with your thumb until it comes off the bottle
4. remove the cannula from the sachet in which it is contained
5. press the cannula into the special opening on the hook side (part without holes) until you hear a "click"

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INFO@THERMALIS.IT — (+39) 041 0996 863

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6. gently insert the cannula into the vagina
7. press the sides of the bottle until it is completely empty.
The patient is advised to irrigate the area, lying or sitting on the bidet or toilet or in the bathtub, trying to keep the solution in the vagina for a few minutes.
Cannulas are for personal use only; if reused, they must be thoroughly washed and disinfected.

Components

- Chamomile flower water, with a soothing effect
- Salso-bromo-iodic Hyperthermal Water, that plays a key role in gynaecological pathologies
- Due to the high saline concentration, it has a positive effect on the urogenital system. When Hyperthermal Water interacts with the mucous membranes, it produces outward serous currents which evacuate germs, inflammatory products, and recall excess liquids from living tissues with which they come into contact (osmotic effect). It also has a soothing effect and respects the vagina's pH physiology.
- Glycogen: a dynamic ecosystem that constantly maintains vaginal health during the natural phases of a woman's life (menstruation, pregnancy and menopause). This ecosystem is made up of healthy micro-organisms that colonize vaginal surfaces with positive bacteria, the most common and beneficial of which are lactobacilli. In women of childbearing age, the presence of lactobacilli helps to produce lactic acid by metabolizing glycogen, thus rebalancing the environmental pH of the natural protective layer to between 3.8 and 4.5. Protective layers offer natural resistance to potential health risks associated with bacterial vaginosis, vaginitis and common anaerobic pathogens. Lactobacilli metabolize glycogen into lactic acid, creating an acidic environment. The acidic nature favors the growth of many Lactobacillus strains and inhibits the growth of harmful bacteria. The natural process is ultimately aided by the female hormone oestrogen, which initiates the release of glycogen from the vaginal. However, throughout life, oestrogen levels change and reduce the amount of glycogen available
- Lactic acid: helps maintain the vagina's physiological acid pH

Warnings

THERMAGYN® Vaginal Douche must be applied before all gynecological treatments (vaginal ovules, vaginal suppositories, etc.) carried out simultaneously. Do not mix the product with other substances or preparations, even of similar use. Do not use on wounds or injuries. During pregnancy, use only after advice from your medical specialist. Do not use for long periods. Do not use in case of hypersensitivity to the product or any of its components. Keep out of reach of children. Do not use it other than intended. Do not use if packaging is not intact or damaged. Keep this leaflet for future reference. Do not dispose of the empty container in the environment. Store the product between 5° C and 25° C, in a dry place away from sources of heat and direct light. Do not use after the expiry date, which refers to the product properly stored.

Packaging

2 disposable bottles of 100 ml + 2 cannulas.

Ingredients

Aqua*, Chamomile Water, Terpinen 4-OL, Lactic Acid, Glycogen, Potassium Sorbate, Hydrogenated castor oil ethoxylate 40 moles, Hydroxyethylcellulose.

*With Salso-bromo-odic Hyperthermal Water

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